

Amalgam Carrier

Instructions for Use:

- It is advisable to lubricate the nozzle with a little Vaseline or silicone spray before replacing the nozzle after autoclaving
- Do not over-tighten the nozzle as this can cause stress at the joint and may weaken it
- The Amalgam Carrier is not designed to be used as a lever, the nozzle may break if put under great stress
- The nozzle may wear in time and has a limited lifespan. Replacement nozzles are available.

Cleaning Instructions:

- Remove nozzle and clean all residue amalgam with an interspace brush
- Clean head of the plunger with a tissue to remove all traces of amalgam
- Autoclave at 134°C for a maximum of 10 minutes

Warning – the amalgam carrier is made of plastic which will begin to deform above temperatures of 150°C